



MADE IN THE USA

Accessories (each sold separately):

- Caddies MP-015 (shown below), MP-042, and MP-085
- Bowman Hangers MB-450, MB-500, MB-510 and MB-600
- Glove Dispenser Add-Ons GD-011, GD-012 or GD-111



Product Description:

- Protection System
- Holds disposable or reuseable gowns, aprons, or coveralls/ jumpsuits, two boxes of gloves, one box of earloop masks, and one box of face shields
- Keyholes for wall mounting, or door hanger cut out for use with wall or door hanger
- Quartz Beige ABS Plastic

Primary Area for Product Usage:

- Anywhere personal protection equipment is needed

Dispenses:

- Left: Box(es) no larger than 14.80"W x 14.73"H x 4.50"D (37.6 cm x 37.4 cm x 4.50 cm)
- Top right: Box(es) no larger than 8.19"W x 5.39"H x 4.48"D (20.8 cm x 13.7 cm x 11.4 cm)
- Middle right: Box(es) no larger than 8.19"W x 9.65"H x 4.48"D (20.8 cm x 24.5 cm x 11.4 cm)
- Bottom: Box(es) no larger than 11.02"W x 5.51"H x 4.02"D (28.0 cm x 14.0 cm x 10.2 cm)


Package Specifications:

- Package Quantity: 1 per case
- 24"W x 8"H x 24"D (61.0 cm x 20.3 cm x 61.0 cm)
- 8.0 lbs (3.6 kg) - approximated

Product Specifications (overall external dimensions):

- 23.50"W x 20.80"H x 4.75"D (59.7 cm x 52.8 cm x 12.1 cm)
- 5.6 lbs (2.5 kg) - approximated

Mounting Specifications:

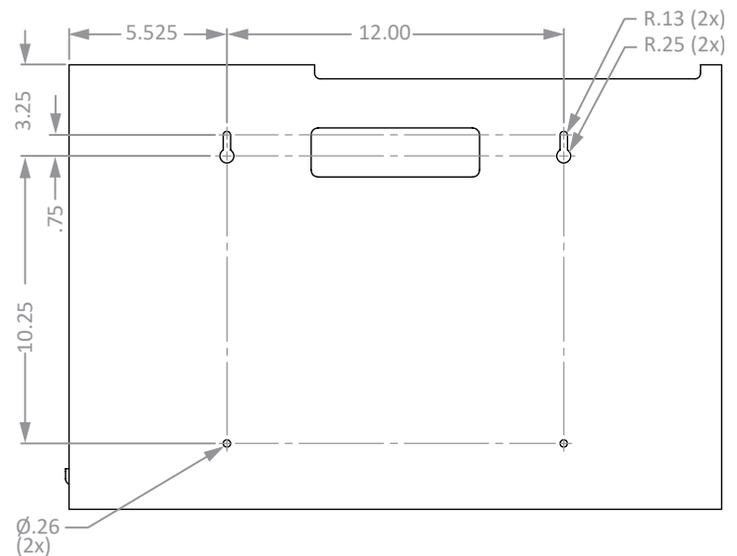




Installation


Always mount dispensers securely *prior* to loading with product.

- Organizer is expandable and additional sections can be purchased to customize as needed.

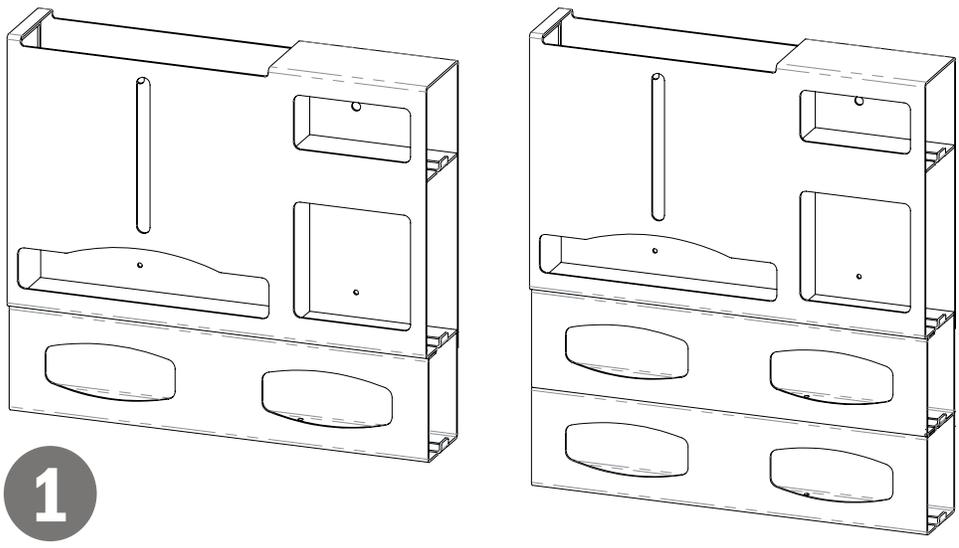

1

- Mount securely to a wall with screws approved and supplied by building maintenance.
- Dispenser can be mounted onto a BOWMAN® door or wall hanger through the cutout in the back. (Please see chosen hanger datasheet for details.)

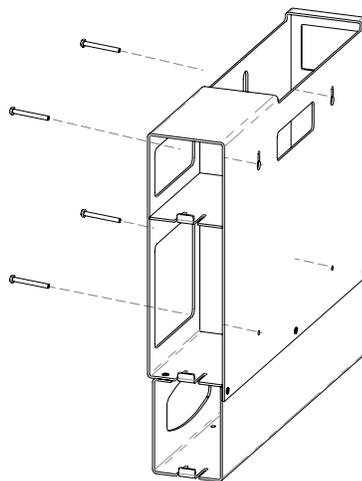

2

Parts included:

- Dispenser

Tools Required:

- No tools required to assemble dispenser

Care and Construction

ABS Plastic

Material Characteristics

Designed for Long-Term, Heavy-Duty Applications

The combination of copolymers (Acrylonitrile, Butadiene, and Styrene) gives ABS an excellent combination of strength, rigidity, and toughness. The fauxwood finish is ARVINYL laminate created from a semi-rigid film.

ABS is chemical and stress-crack resistant. It is tough, dimensionally stable, and will hold up to long-term use. To ensure product longevity, fauxwood products should not be mounted in very hot or cold areas.

Care and Cleaning

Use mild soap and water to clean. Petroleum-based oils, solvents, and paints should not be used.

Standard hospital cleaning supplies are approved for use on our ABS plastic products.